



USER MANUAL

Full Electric LOW Home Care Bed

3 Function electric bed, hand pendant raises the head, foot and height of the bed.

w/ Half Rails - Model # PMFEBLHR-BLK

w/ Full Rails - Model # PMFEBLFR-BLK

without Rails - Model # PMFEBL-BLK

Semi Electric Home Care Bed

2 Function electric bed, hand pendant raises the head and foot of the bed.

w/ Half Rails - Model # PMSEBHR-BLK

w/ Full Rails - Model # PMSEBFR-BLK

without Rails - Model # PMSEB-BLK

IMPORTANT: CHECK THE LABEL ON THE BED FRAME FOR THE CORRESPONDING MODEL NUMBER

DEALER: This manual must be given to the user.

USER: please read this manual completely for warnings, proper assembly, maintenance, and safe usage instructions before using this bed. Save manual for future reference.

CAUTION: DO NOT attempt to operate bed controls prior to completion of assembly as damage to bed components may occur. In case of damage DO NOT use.

590 Franklin Ave, Mount Vernon, NY 10550

855-237-7622

www.proactivemedical.com

Special Notes

NOTICE

The information contained in this document is subject to change without notice. Check all parts for shipping damage and test before using. In case of damage, DO NOT USE. Contact dealer for further instruction.

WARNING

Your electric bed has been totally engineered to provide you with reliable operation and the strength you deserve. The bed has been thoroughly tested and inspected prior to shipment. The utmost in comfort and safety has been provided for you. The bed is for HOME use only. It is NOT for hospital use and was not designed to meet hospital standards.

DO NOT use near explosive gases. Possible fire hazard when used with oxygen administering equipment other than nasal or masked type.

When using nasal or masked type oxygen administering equipment, oxygen and/or air tubing MUST be routed and secured properly to ensure it does NOT become entangled and eventually severed during normal operation of the bed.

WARNING

When using liquids in or around the bed use caution to be sure that liquids of any kind are not spilled. If a liquid is spilled in or around the bed, UNPLUG the bed immediately. Clean up spill and allow bed, and/or area around bed, to dry thoroughly before using the electric controls again.

Close supervision is necessary when this product is used by or near CHILDREN OR PHYSICALLY CHALLENGED INDIVIDUALS.

This product should NEVER be left unattended when plugged in.

NEVER PERMIT ANYONE UNDER THE BED AT ANY TIME.

When operating/moving electric beds, ALWAYS ensure that the individual utilizing the bed is positioned properly within the confines of the bed. DO NOT let any extremities outside the bed frame or between the bed rails when performing these functions.

WARNING

DO NOT OPERATE THIS EQUIPMENT WITHOUT FIRST READING AND UNDERSTANDING THE WARNINGS, CAUTIONS AND INSTRUCTIONS IN THIS MANUAL. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS AND INSTRUCTIONS, CONTACT A HEALTHCARE PROFESSIONAL, DEALER OR TECHNICAL PERSONNAL IF APPLICABLE BEFORE ATTEMPTING TO USE THIS EQUIPMENT, OTHERWISE INJURY OR DAMAGE MAY RESULT.

TO AVOID POSSIBLE INJURY OR DEATH, USE A SIDERAIL BUMPER PAD SO THAT THE GAP BETWEEN THE TOP OF THE MATTRESS AND THE BOTTOM OF THE SIDE RAIL IS SMALL ENOUGH TO PREVENT A PATIENT FROM GETTING THEIR HEAD/NECK BETWEEN THE MATTRESS AND SIDE RAIL AND POSSIBLY SUFFOCATING.

Table of Contents

Special Notes	2
Table of Contents.....	3
Safety Instructions.....	4
Assembly Instructions.....	6
Motor Installation.....	11
Bed Rail Installation.....	12
Optional Fixed Height Adjustment.....	14
Operation and Adjustment.....	15
Maintenance and Safety Checks.....	19
Specifications.....	20
Warnings.....	22
Warranty Registration.....	26

Safety Summary

Operating Information

WARNING

After bed has been assembled, always test to make sure that all sections of the bed are properly and securely in place before using.

Make sure head and foot springs are securely locked to the head and foot boards before use. Pendant cord **MUST** be routed and secured properly to ensure that the cord does NOT become entangled and severed during operation of Semi Electric or Electric bed.

Keep all moving parts free of obstruction (i.e. blankets/sheets, heating blankets/pads, tubing, wiring, etc. and other types of products using electric cords which may get tangled around the be, side rails or legs during operation of Electric beds:

- Mainframe and mattress deck (head/foot deck)
- All drive shafts

Electric beds are equipped with wheel locks. Be sure to apply them before attempting any transfer. Inspect for correct locking action before actual use. When leaving Electric Bed, always engage wheel locks. Also be certain the wheel locks are activated to prevent the wheels from moving.

The side rails when used with an electric bed **DO NOT** fall within any weight limitations. Side rails can be deformed or broken if excessive side pressure is exerted on the side rails. These side rails are for the purpose of preventing an individual from rolling/climbing out of bed.

If necessary, a physician or a healthcare professional should be consulted for alternative means of safe restraint. The side rails are **NOT** intended nor may be used for restraint purposes.

DO NOT use the side rails as push handles for moving the bed.

Trapeze units must be positioned on either side of the drive shaft (head/foot as near as possible to the center point of the bed. The trapeze is to be used **ONLY** IN **ASSISTING** the patient in repositioning or transferring in or out of bed.

Traction units **MUST** be securely fastened to head/foot ends of the bed. These units are to be used **ONLY** for immobilizing a patient in various traction setups or **ASSISTING** the patient in repositioning or transferring in or out of bed.

- Never allow patients to use the trapeze or traction units as a total individual weight support.
- Make certain ALL screws and nuts are tight at all times.
- Never permit more than one (1) person on/in the bed at any time.
- Body weight should be evenly distributed over the surface of the bed. **DO NOT** lay, sit or lean in such a way that your entire body weight is placed on **RAISED** head or foot sections of the bed. This includes when assisting in repositioning or transferring in or out of bed.

REPLACEMENT PARTS

Replacement mattresses and bed rails with dimensions **DIFFERENT** than the original equipment supplies or specified by the bed frame manufacturer may not be interchangeable. Variations in bed side rail design and thickness or density of the mattress could cause entrapment. **USE ONLY** Proactive Medical replacement parts.

Safety Summary - continued

ELECTRICAL - Grounding Instructions:

Some devices equipped with three-prong (grounding plugs for protection against possible shock hazards. Where a two-prong wall receptacle is encountered, it is the personal responsibility and obligation of the customer to contact a qualified electrician and have the two-prong receptacle replaced with a properly grounded three-prong wall receptacle in accordance with the National Electric Code. If you must use an extension cord, use ONLY a three wire extension cord having the same or higher electrical rating as the device being connected. DO NOT, under any circumstances, cut or remove the grounding prong from any plug used with or for Proactive Medical products. In addition, Proactive Medical has placed red WARNING tags on some equipment. DO NOT remove these tags.

Never operate if the unit has a damaged cord or plug. If it is not working properly, call a qualified service professional for examination and repair. Keep the cord away from HEATED or HOT surfaces.

REPAIR OR SERVICE INFORMATION

DO NOT open assemblies such as the motors, hand control unit, junction boxes or electrical control boxes. ONLY trained service personnel certified by Proactive Medical are qualified to repair these parts. If an uncertified individual performs any work on these beds, the warranty is voided. DO NOT use parts, accessories, or adapters other than those distributed by Proactive Medical. UNPLUG the power cord when performing any maintenance on the bed.

RADIO FREQUENCY INTERFERENCE

Most electronic equipment is influenced by Radio Frequency Interference (RFI). Caution should be exercised with regard to the use of the portable communications equipment in the area around such equipment. If RFI causes erratic behavior, unplug the bed immediately. Leave unplugged while the transmission is in progress.

WEIGHT LIMITATION

The weight limitation for Proactive Medical 36-inch wide Electric Bed - taking into consideration patient weight, mattress, rails, proper positioning, realignment, transfer and overall care is 450lbs.

Assembly Instructions

1. Confirm that all parts are present:

- A. Foot Deck Section
- B. Head Deck Section
- C. Headboard & Footboard

- D. Casters (4) two locking, two non-locking
- E. Hand Crank
- F. Hi/Low Drive Shaft
- G. Head/Foot Deck Section Motors with Hand Control





Caution: DO NOT attempt to operate bed controls until completion of assembly as damage to bed components and/or personal injury may occur.


Assembly Instructions - continued

FRAME ASSEMBLY

2. Place both ends of the frame/sleep surface on their side and align the flange with the receptacle as shown.

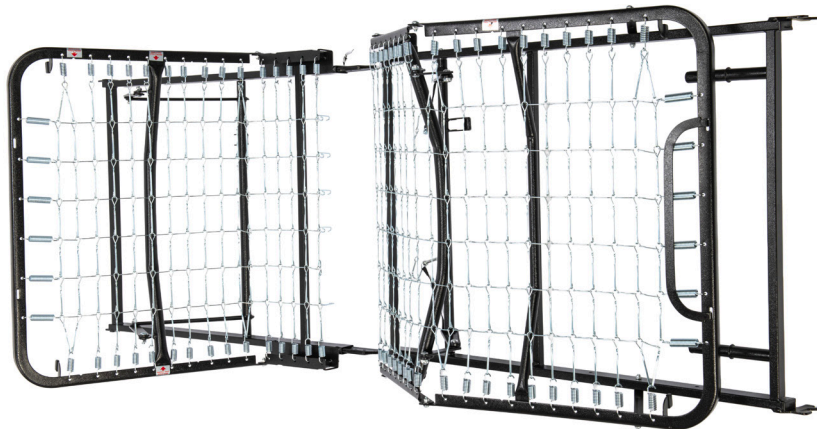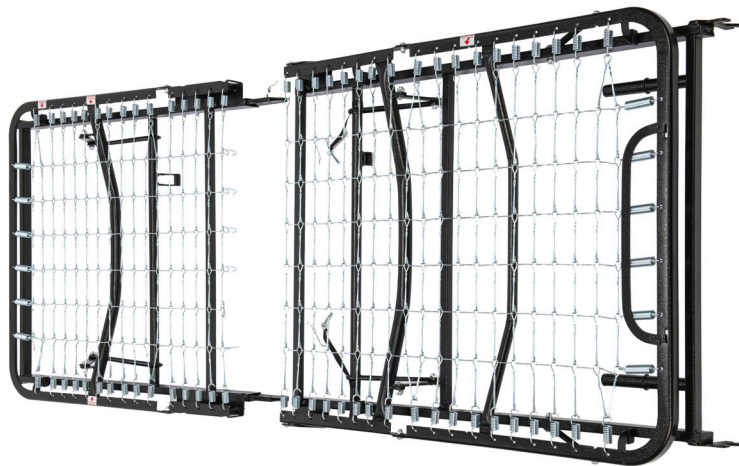

Assembly Instructions - continued

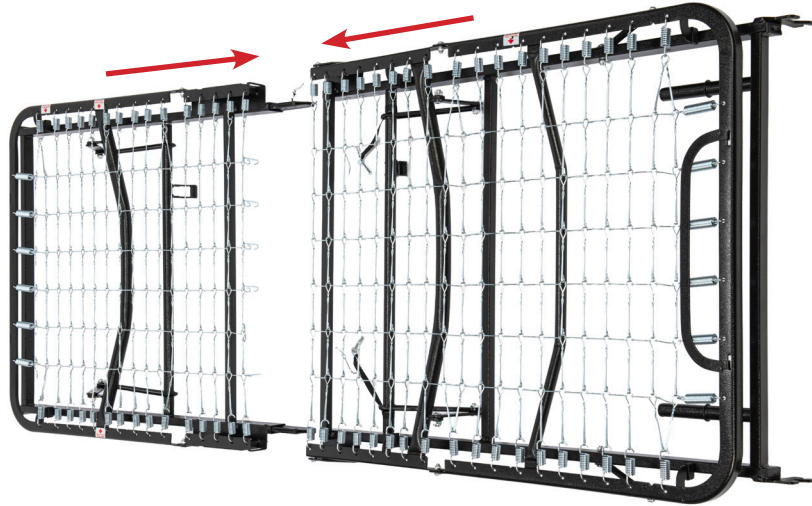

3. To attach the head and foot sections, slide the two sides together and lock into position with cotter pin and clasp.



To disassemble the head and foot section, remove cotter pin and clasp and separate sections.

Assembly Instructions - continued

4. Secure the frame/sleep surface with the cotter pin as shown (Figure A). Lock the cotter pin with the attached clasp. Repeat on the opposite side (Figure B).



Figure A

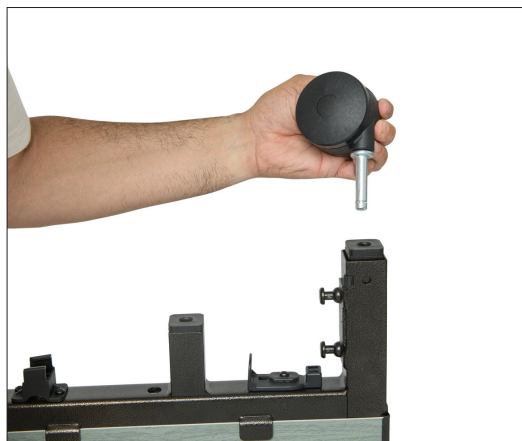


Figure B

Assembly Instructions - continued



Wheel placement for low position setup




Wheel placement for height adjustable setup



5. Assemble Bed Ends to Frame:

Install casters onto bed ends by inserting the stem of the caster into the leg receptacle. It is recommended to install one locking caster and one non-locking caster on each bed end. The locking casters should be installed diagonally from each other. Install casters to both bed ends. To engage locking caster, press down on locking tab; lift up to release.

Attach the Head and Foot Boards by aligning the frame mounting hooks over bed end mounting pins and push frame down to secure in place. Push Slide Locks in place over frame hooks for both bed ends to help prevent frame separation from bed ends.

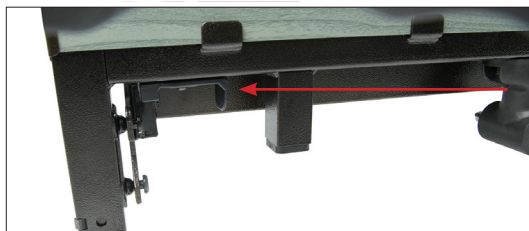

Install the head end of the shaft as shown.



Assembly Instructions - continued



NOTE: Before installing motor, plug hand pendant and Hi/Low motor cable into motor as shown.




Align the motor with the actuator bar and lift the head spring to the highest position. The motor will automatically be pulled towards the bed and lock onto the actuator. Repeat for foot section



After the motor is installed, slide locking covers back on motor.

Assembly Instructions - continued

Attaching Bed Rails *if included w/model purchased

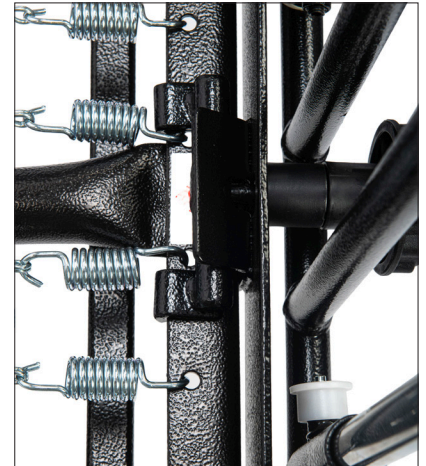

NOTE: Half Rails
The head section features labels on the frame marked, "Attach Half Rails Here".



NOTE: Full Rails
The foot and head bed frame sections have labeled notches to attach spring loaded full rail cross bars.

Assembly Instructions - continued

BED RAIL ASSEMBLY

Full Bed Rails:

To install Full Bed Rails, place the 2 full rail cross bars into the label notches in the frame to receive the full rail posts into the receptacle in the vertical position. Repeat for all four receptacles.



A



B



Full Bed Rail Operation: Pull out Pull Pin on each bracket one at a time to raise or lower rail. Ensure Pull Pins are properly engaged to lock rails in position.

Assembly Instructions - continued



Optional fixed height adjustment with lock pins.



NOTE: The four large pins are used if the height of the bed is to be locked at a set height. Set the height of the bed so that the hole in the leg is aligned with the hole in the bed end outer leg. Insert the pin through each of the four legs and install the spring clip on each.



Before a mattress is laid on the bed, flip up the mattress guard to reduce the movement of the mattress.

Operation and Adjustment - Full Electric Beds

This Procedure Includes the Following: Full Electric Operation Semi-Electric Operation

Full Electric Operation

1. The accompanying pictures detail the full-electric beds and will acquaint the user with the bed functions and controls.
2. Operate the bed through all phases of its operation.
3. If any problems arise during test, recheck all electrical connections and mechanical hook ups.
4. Full-Electric beds use a six (6) function pendant for all bed operation.
5. All Full-Electric beds come with an emergency crank to allow continued operation in the event of a power outage.



Full electric bed in height adjustable position.



Full electric bed in low position.

Raising and lowering the entire bed Full-Electric Beds

TO RAISE ENTIRE BED FULL ELECTRIC BEDS

Depress the
UP button as
shown on
pendant
control.


This will raise
the entire
height of the
bed.



TO LOWER ENTIRE BED FULL ELECTRIC BEDS

Depress the
DOWN
button as
shown on
pendant
control.

This will lower
the entire
height of the
bed.

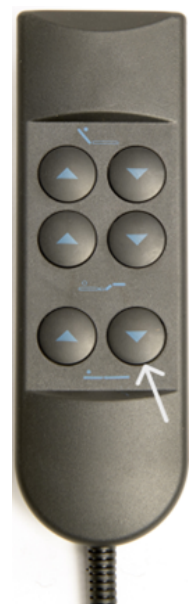

Operation and Adjustment - Full Electric Beds

Raising and lowering the head and foot sections bed Full-Electric Beds

TO RAISE THE HEAD SPRING SECTION FULL ELECTRIC BEDS

Depress the
UP button
shown on
pendant
control.

This will raise
the head
portion
ONLY.



TO LOWER THE HEAD SPRING SECTION FULL ELECTRIC BEDS

Depress the
DOWN
button shown
on pendant
control.

This will lower
the head
portion
ONLY.



TO RAISE THE FOOT SPRING SECTION FULL ELECTRIC BEDS

Depress the
UP button
shown on
pendant
control.

This will raise
the foot
portion
ONLY.



TO LOWER THE FOOT SPRING SECTION FULL ELECTRIC BEDS

Depress the
DOWN
button shown
on pendant
control.

This will lower
the foot
portion ONLY.



Operation and Adjustment - Semi-Electric Beds

Semi-Electric Operation

1. The accompanying pictures detail the semi-electric beds with the hi/lo and foot crank locations where applicable.
2. The semi-electric single and two crank beds with hi/lo operation use a four (4) function pendant for head and footspring functions and a hand crank for hi/lo bed operation.
3. The semi-electric fixed height beds will feature head and foot section operation but the overall bed height is fixed and cannot be altered.
4. Determine your particular semi-electric bed to assist in the operation of the features of the bed.
5. Operate the bed through all phases of its operation.
6. If any problems arise during test, recheck all electrical connections and mechanical hook ups.
7. All semi-electric beds come with an emergency crank to allow continued operation in the event of a power outage.

Raising and Lowering the Entire Bed Semi-Electric Beds

TO RAISE ENTIRE BED SEMI-ELECTRIC BEDS



Semi-Electric Single Crank - Hi/Lo
Turn crank clock wise. This will raise
the entire height of the bed.

Turning crank counter clockwise will
lower the entire height of the bed.

TO RAISE AND LOWER THE HEAD SPRING SECTION - SEMI-ELECTRIC BEDS

Depress the UP
button shown on
pendant control.

This will raise the
head portion
ONLY.

Depress the
DOWN button
shown on
pendant control.

This will lower the
head portion
ONLY.



TO RAISE AND LOWER THE FOOT SPRING SECTION - SEMI-ELECTRIC BEDS

Depress the UP
button shown on
pendant control.

This will raise the
foot portion
ONLY.

Depress the
DOWN button
shown on
pendant control.

This will lower the
foot portion ONLY.



*NOTE: Remember that in the event a full or semi-electric bed should malfunction, it is possible to manually crank the bed using the emergency crank supplied.

Operation and Adjustment - Semi-Electric Beds

TO RAISE OR LOWER BOTH THE FOOT AND HEAD SPRING SIMULTANEOUSLY - SEMI-ELECTRIC BEDS

Depress the UP
buttons shown
on the pendant
control.

This will raise both
the head and foot
portion at the
same time.

Depress the
DOWN buttons
shown on
pendant control.

This will lower both
the head and foot
portion at the
same time.



Maintenance and Safety Checks

Proactive Medical recommends the following maintenance and cleaning procedures be conducted between users.

ELECTRONICS - Every six months each new user.

Check all controls to make sure all functions work properly.

- Foot control
- Head control
- Hi/Lo

Check all cables for damaged or frayed wires.

- Power cord
- Pendant cord

Check to make sure all plugs are fully inserted or attached.

Check 9V battery on motor and replace annually.

BEDFRAME & SLEEPING SURFACE - One year or each new user.

Visually check all welds.

- Head section
- Foot section
- Main frame

Check joints between sleeping surface sections for loose fasteners.

CLEANING - Each new user

- The metal parts of the bed are covered with a powder coating. Clean all coated parts with mild detergent and warm water. Periodically raise head and feet sections of the bed and remove dust from frame. Also, periodically remove mattress and clean mattress deck.

LUBRICATION & MECHANICAL - Six months or each new user.

- Lightly grease all actuator screw threads with white lithium grease.
- Lubricate all caster roller and swivel bearings with light machine oil.
- Check all bolts and tighten as needed.

WARRANTY

- Limited Lifetime warranty on frame
- 5 years on motor

During the warranty period, defective items will be repaired or replaced at Proactive Medical's option at no charge.

Specifications

Deck Height

Bed Height
in height adjustable
position



High Position

23"

Low Position

15"

Bed Height
in low position



9.25"

Bed Length



80"

Bed Width

36"

Electrical

- Motor is UL approved

Weight

- Normal operating conditions: 450lbs evenly distributed.

Specifications - continued

Specifications for Bed Model # PMFEBLHR-BLK & PMFEBLFR-BLK



The lowest position with the inner castor setting is 9.25"

The lowest position with the outer castor setting is 10.24"



The highest position: 21.06"



GENERAL WARNINGS

DO NOT use this product or any available optional equipment without first completely reading and understanding and reading these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact healthcare professional, dealer, or technical personnel before attempting to use this equipment. Otherwise, injury or damage may occur.

Read all the instructions before using the bed.

Refer to the owner's manuals for beds and rails for additional product and safety information.

After any adjustments, repair or service and before use, make sure all attaching hardware is tightened securely.

After the bed has been assembled, always test to make sure that all sections of the bed are properly and securely in place before use.

ALWAYS use caster locks except when moving the bed.

Bodyweight should be evenly distributed over the surface of the bed. DO NOT lay, sit, or lean in such a way that your entire body weight is placed only on the raised head or foot sections of the bed. This includes when assisting the user in repositioning or transferring in or out of bed. Check all parts for shipping damage and test before using in case of damage, DO NOT use it. Contact a qualified technician for further instruction. Close supervision is necessary when this bed is used by or near children or people with disabilities.

DO NOT let any individual underneath the bed or in between the raised bed frame components at any time. DO NOT permit more than one person on/in the bed at any time.

DO NOT use outdoors. DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand

the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

For Dealers Only - Set-up and Assembly Instructions are in the rear of this manual. These procedures must be performed by a qualified technician only. The March 2006 version of the FDA's bed safety guidelines are published by Hospital Bed Safety Workgroup. The latest revision of this document is available at <http://www.fda.gov>.

If the unit is not working properly, call a qualified technician to examine the unit and repair it.

Proactive Medical products are specifically designed and manufactured for use in conjunction with Proactive Medical accessories. Accessories designed by other manufacturers have not been tested by Proactive Medical and are not recommended for use with Proactive Medical products.

Keep all moving parts, including the main frame, mattress deck (head and foot springs/sections) and all drive shafts free of obstruction (i.e. blankets/ sheets, heating blankets/pads, tubing, wiring, etc. and other types of products using electric cords which may get tangled around the bed, side rails or legs) during operation of the bed.

Keep the product a minimum of 12 inches away from any direct heat source.

Make sure head and foot springs/sections are connected securely to the universal bed ends before use.

Physically challenged individuals who cannot prevent themselves from rolling/climbing out of the bed may require alternative safe means of restraint.

Procedures other than those described in this manual must be performed by a qualified technician.

WARNING: RISK OF INJURY TO PERSONS!

DO NOT place video equipment such as televisions or computer monitors on bed.

NEVER allow patients to use trapeze or traction units as a total individual weight support.

Trapeze units are to be used only for assisting the patient in repositioning or transferring into or out of bed.

Trapeze units must be positioned on a universal bed end as near as possible to the center point of the bed end.

The bed is not designed to be used as a patient transport. When transporting a patient, use an approved patient transport. Otherwise, injury or damage may result.

The bed may be equipped with locking casters. When transferring into or out of the bed, always lock the locking caster(s). Inspect for correct locking action on caster locks before using bed. Even with casters properly locked, some flooring surfaces such as tile or wood, will allow the bed to move under some conditions. Use on surfaces such as these must be evaluated by the care provider.

The initial set up of this bed must be performed by a qualified technician.

The total weight limit of the Proactive Medical 36-inch wide Electric bed (including accessories, mattress, occupant, and any other person/object positioned on the bed) is 450 pounds; 350 pounds patient weight.

Use this bed only for its intended use as described in these instructions. DO NOT use attachments not recommended by the manufacturer.

After raising/lowering the head/foot end of the bed, check the distance between the bottom of the bed rail and the mattress. If there is excessive distance between the bottom of the bed rail and the mattress in which individuals may become entangled, adjust the height of the bed rail (if applicable), or provide alternative means of patient protection.

Once patient assessment concludes that the patient's condition increases the chance of entrapment, the bed MUST be in the flat position when left unattended. Proper patient assessment and monitoring, and proper maintenance and use of equipment is required to reduce the risk of entrapment. Variations in bed rail

dimensions, and mattress thickness, size or density could increase the risk of entrapment.

Visit the FDA website at <http://www.fda.gov> to learn about the risks of entrapment. Replacement mattresses and bed side rails with dimensions different than the original equipment supplied or specified by the bed frame manufacturer are not interchangeable. Variations in bed side rail design, width, and thickness or firmness of the mattress could cause/contribute to entrapment. Use only authorized Proactive Medical replacement parts and/or accessories otherwise the warranty is void. Proactive Medical will not be responsible for any damage or injury that may result.

To reduce the risk of entrapment, make certain that the bed rail cross braces DO NOT exceed the width of the mattress. The mattress MUST fit bed frame and side rails snugly to reduce the risk of entrapment.

When operating this bed, ALWAYS ensure that the individual utilizing the bed is positioned properly within the confines of the bed. DO NOT let any extremities protrude over the side or between the bed rails when performing these functions. Proactive Medical recommends that the mattress be centered on the bed frame. Otherwise, individuals may become entangled between the bed rail and the bed frame.

Mattress MUST fit bed frame and assist rails snug to reduce the risk of entrapment.

NOTE: When using nasal or masked type oxygen administering equipment, the oxygen or air tubing MUST be routed and secured properly to ensure that the tubing does not become entangled and/or severed during normal operation of the bed.

SAVE THESE INSTRUCTIONS.

NOTE: Updated versions of this manual are available on www.proactivemedical.com.

BED RAIL WARNINGS

Although bed rails are not rated to any specific weight limitation, the bed rails may become deformed or broken if excessive side pressure is exerted on the bed rails. The bed rail is not an assist rail for getting into or out of bed. DO NOT use the bed rails as push handles when moving the bed.

Always test to make sure that the side rails are properly and securely in place before using the bed.

DO NOT install the optional bed rails without reading and understanding all the instructions in the instruction sheet that accompanies the bed rail kit.

DO NOT use the side rails as push handles for moving the bed.

These bed rails are intended to prevent an individual from inadvertently rolling out of bed. DO NOT use for restraint purposes.

When used with a manual/electrical bed, the bed rails DO NOT fall under any weight limitations. Bed rails can be deformed or broken if excessive side pressure is exerted on the bed rails. These bed rails are used for the purpose of preventing an individual from inadvertently rolling out of bed. The bed rails are NOT intended nor may be used for restraint purposes. If an individual can injure himself/herself, a physician or a healthcare professional should be consulted for alternative means of safe restraint.

ELECTRICAL WARNINGS

A safety feature of this product includes protection against overheating caused by excessive or extended periods of operation. Depending on the duration, this includes multiple or repeated adjustments or the use of multiple functions at one time. To ensure trouble free operation, always allow a slight pause between multiple adjustments and avoid pressing more than one function button at a time. If thermal protection activation should occur, the bed will not respond to pendant commands.

Given this situation, release the pendant button and allow the bed unit to sit for several minutes.

This will allow the protection function time to reset and restore bed function. Depending on severity of the initial overheating, this could take up to 30 minutes.

Allow a slight pause between adjustments and avoid pressing multiple buttons at the same time. If pendant buttons are depressed too rapidly or multiple buttons are pressed at the same time, the desired feature may not activate. Simply release the pendant button, permit a slight pause, and then activate the next operation.

ALWAYS remove manual crank(s) before performing electronic functions. Otherwise, the crank(s) will turn when the motor is on and could cause personal injury or damage to the bed. Always unplug the bed from the electrical outlet before cleaning.

DO NOT open assemblies such as the motors, pendant, junction boxes or gear boxes. No user serviceable parts are inside. Only qualified technicians are permitted to repair these parts. If unqualified individuals perform any work on these beds, the warranty is void.

DO NOT place pendant under or between objects. This may unintentionally press the buttons and may cause injury or damage.

Electronic equipment may be influenced by Radio Frequency Interference (RFI). Caution should be exercised regarding the use of portable communications equipment in the area around such equipment. If RFI causes erratic behavior, unplug the electric bed IMMEDIATELY. Leave unplugged while transmission is in progress.

Ensure all cables and cords are routed such that they will not become entangled or pinched. Otherwise, damage or injury may result. If a liquid is spilled in or around the electric bed, unplug the electric bed before cleaning. Clean up the spill and allow the electric bed or the area around the electric bed to dry thoroughly before using the electric controls again. Inspect the covering of the bed's control panel and the patient control panel to assure that the covering is not cracked or damaged.

Keep all electrical cords away from heated or hot surfaces.

Keep the cord away from heated surfaces. NEVER operate if the unit has a damaged cord or plug. If it is not working properly, if it has been dropped, damaged, or dropped into water. Return the bed to a service center for examination and repair.

On Full Electric bed, the Hi/Lo crank MUST be removed before the bed is used. Failure to remove the crank may cause damage or personal injury.

Refer servicing to qualified personnel only. Grounding reliability depends upon a properly grounded wall outlet. The pendant and power cords must be routed and secured properly to ensure that the cords DO NOT become entangled, pinched and/or severed during operation of the electric bed.

Unplug from the outlet before servicing. Unplug the power cord from its power source before performing any maintenance on the Electric bed. DO NOT unplug the power cord from the junction box. Damage to cord will result.

Use caution when disconnecting the pendant. DO NOT press pendant buttons.

When bed is not to be used for an extended period, unplug electric bed from the wall outlet.

When using an extension cord, use only a three-wire extension cord having at least 16 AWG (American Wire Gauge) wire and the same or higher electrical rating as the device being connected. Use of improper extension cord could result in a risk of fire and electric shock. Three to two prong adapters should not be used. Use of three prong adapters can result in improper grounding and present a shock hazard to the user.

DO NOT use near explosive gases.

Possible fire hazard when used with oxygen administering equipment other than nasal or masked type.

Use masked or nasal type oxygen administering equipment only in conjunction with this bed. The use of ANY other type of oxygen administering equipment can result in a fire hazard.

Bodyweight should be evenly distributed over the surface of the bed.

DO NOT lay, sit or lean in such a way that your entire body weight is placed only on raised head or foot sections of the bed. This includes when assisting the user in repositioning or transferring in or out of bed.



The Head (Head Spring) and Foot (Foot Spring) sections of the sleep surface must be in the lowered/flat position for entry into the bed. Patients should be slowly lowered into the bed. DO NOT abruptly "drop" patients into the bed. It is not safe to use the Head (Head Spring) or Foot (Foot Spring) sections of the sleep surface to assist in raising or lowering patient for entry or egress from the bed. This may damage the frame and/or motors and create a danger to the patient. DO NOT attempt to raise the Head (Head Spring) sections of the sleep surface while the patient or any other individual is seated or positioned with their weight entirely on the Head (Head Spring) section of the sleep surface.



DO NOT attempt to raise the Foot (Foot Spring) section of the sleep surface while the patient or any other individual is seated or positioned with their weight entirely on the Foot (Foot Spring) section of the sleep surface.

DO NOT attempt to raise the Head (Head Spring) or Foot (Foot Spring) sections of the sleep surface while more than one person is lying in the bed or when the bed weight limitations have been exceeded.

Proactive Medical Homecare Bed Warranty

This warranty is extended only to the original purchaser who purchases this product when new and unused from Proactive Medical or a dealer. This warranty is not extended to any other person or entity and is not transferable or assignable to any subsequent purchaser or owner. Coverage under this warranty will end upon any such subsequent sale or other transfer of title to any other person.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Proactive Medical warrants the bed frame and electronics when purchased new and unused to be free from defects in materials and workmanship as follows;

Limited lifetime on frame, 5 years on electronics, 1 year on pendant from the date of purchase from Proactive Medical or a dealer, with a copy of the seller's invoice required for coverage under this warranty. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Proactive Medical's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Proactive Medical's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Proactive Medical product or Proactive Medical customer service only if purchased from Proactive Medical direct. Provide model number, serial number and the date of purchase and indicate nature of the defect.

DO NOT return products to Proactive Medical without prior consent. Please obtain a return authorization number from Proactive Medical. C.O.D. shipments will be refused; return shipping charges must be prepaid. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty days of return authorization date.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE INSTRUCTION MANUAL THAT COMES WITH THE PRODUCT. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT PROACTIVE MEDICAL'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF PROACTIVE MEDICAL; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND PROACTIVE MEDICAL'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN PROACTIVE MEDICAL DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY PROACTIVE MEDICAL.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. PROACTIVE MEDICAL SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER. THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.